

## Instructions for use

### Comfort Shield® MDS®

(Hylan A fluid, 0.15 %)



### Description


**Comfort Shield** is a sterile, preservative free, viscoelastic, clear shield for the cornea. It consists solely of substances that are natural components of the eye. Therefore, **Comfort Shield** does not create any allergies and can also be used by allergic persons. **Comfort Shield** imitates the function of the natural tears, protects and moistens the eye surface and lubricates it. This provides long-lasting relief for eyes that are dry due to:

- Keratoconjunctivitis sicca (Sjögren syndrome)
- Injuries to the cornea, erosion, corneal surgery
- Wearing contact lenses
- Hormonal disturbances, menopause
- Air conditioning, wind, cold, dry heat, dust, smoke, air pollution
- Work at a computer screen, long car driving, long-haul flights
- Swimming in chlorinated water

### Ingredients

0.15 % Hylan A, sodium chloride, sodium hydrogen phosphate, sodium dihydrogen phosphate and purified water

### Instructions for use

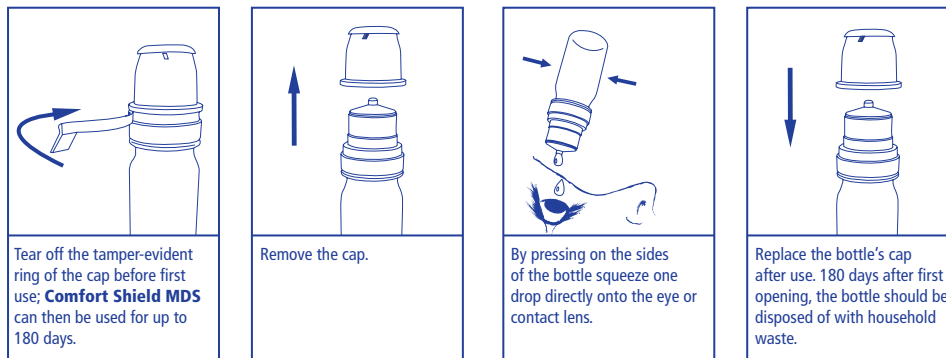

By gently massaging the eyelids with the eyes closed, the product will spread more quickly across the surface of the eye after application.

### Storage

Store between 2°C and 25°C. Protect from frost.

### Precautions and warnings

- Do not use if the tamper-evident ring of the cap or the bottle is damaged.
- Do not allow the bottle to touch your eye.
- Because of the risk of contamination, avoid contact the tip of the bottle with other surfaces or your hands.
- Do not use at the same time with other topical eye medications.
- Do not apply to infected eyes.
- After use, visual acuity may be reduced temporarily; in this case do not drive or operate machinery and wait until normal vision is restored.
- Write down the date of first opening of the bottle on the bottle.
- 180 days after the tamper-evident ring of the cap has been removed the bottle should be disposed of with household waste.

### Pack size

10 ml multidose system (MDS)

### Manufacturer

i.com medical GmbH  
Kafflerstr. 15, 81241 Munich, Germany  
www.icom-medical.de

