

Instructions for use

Comfort Tears® MDS®

(Sodium Hyaluronate, 0.15 %)



Description


Comfort Tears is a sterile, preservative free, viscoelastic, clear liquid for the cornea. It contains only substances that exist naturally in the eye. **Comfort Tears** protects and moistens the eye surface and lubricates it. This provides long-lasting relief for eyes that are dry due to:

- Dry eye disease
- Wearing contact lenses
- Air conditioning, wind, cold, dry heat, dust, smoke, air pollution
- Work at a computer screen, long car driving, long-haul flights

Ingredients

0.15 % sodium hyaluronate, sodium chloride, sodium hydrogen phosphate, sodium dihydrogen phosphate and purified water

Instructions for use

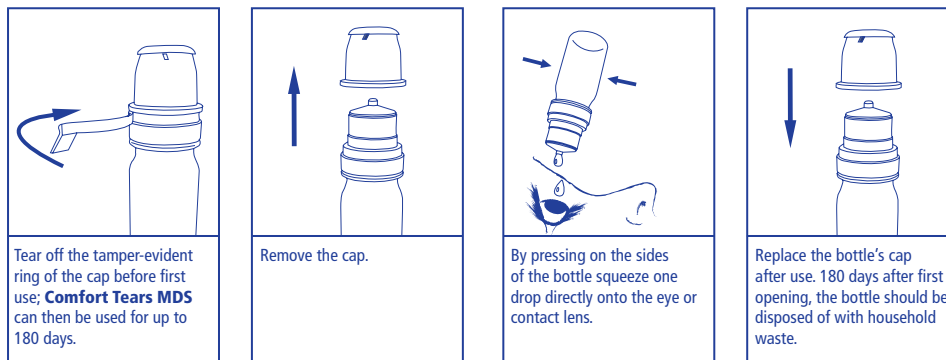

By gently massaging the eyelids with the eyes closed, the product will spread more quickly across the surface of the eye after application.

Storage

Store between 2°C and 25°C. Protect from frost.

Precautions and warnings

- Do not use if the tamper-evident ring of the cap or the bottle is damaged.
- Do not allow the bottle to touch your eye.
- Because of the risk of contamination, avoid contact the tip of the bottle with other surfaces or your hands.
- Do not use at the same time with other topical eye medications.
- Do not apply to infected eyes.
- After use, visual acuity may be reduced temporarily; in this case do not drive or operate machinery and wait until normal vision is restored.
- Write down the date of first opening of the bottle on the bottle.
- 180 days after the tamper-evident ring of the cap has been removed the bottle should be disposed of with household waste.

Pack size

10 ml multidose system (MDS)

Manufacturer

i.com medical GmbH
Kaflerstr. 15, 81241 Munich, Germany
www.i.com-medical.de

